



Advanced Hemostatic Dressing For Ear, Nose and Throat

Instructions for Medical Professionals

Sealk™ is a surgical grade biocompatible hemostatic matrix made from 100% purified, water soluble, etherified natural sodium carboxymethyl cellulose. When Sealk comes in contact with blood or exudate, it quickly transforms into a translucent gel, seals the wound, minimizes blood loss and saves operation time. The translucent gel allows doctor to easily monitor the wound. It also creates a natural moist environment to promote healing.

Indications for Use:

- Emergency and therapeutic use in the control of bleeding from the skin and other surface wounds where temporary control of bleeding is required.
- For wound management when a moist environment is needed.
- For the general population and (EU countries only) for those on anticoagulant medications.

General Use

1. Cut Sealk to the appropriate size and apply directly to the bleeding surface.
2. Apply a second piece if additional protection is desired.
3. Rinse away with sterile saline once the bleeding has stopped.

Post-Op Nasal Packing

1. Select proper size of Sealk. Use dry forceps to pack Sealk into the nasal antrum. Apply pressure to affected arteries.
2. After the bleeding stops completely, Sealk gel will flow out with normal nasal drainage without disrupting the clot. If dry material remains, hydrate and flush out with sterile saline spray.

Post-Op Tonsil Application

1. Apply one or more layers of Sealk to the tonsil bed. Irrigate the area if necessary. Sealk will transform into a gel and adhere to the tonsil bed.
2. Once bleeding has stopped, the excess Sealk gel will gradually dissolve in saliva, without disrupting the clot.

WARNINGS:

- Sealk is intended to assist in the control of wound surface bleeding in cases where hemostasis by clotting is indicated.
- Longer compression times may be required for patients who are obese, hypertensive, or are on anticoagulants.
- When removing Sealk and/or any covering dressing, be careful not to disturb the clot.
- Never peel Sealk from a wound, as re-bleeding could occur. If Sealk dries out, soak thoroughly to rehydrate before rinsing it off.
- In case of swelling or redness, discontinue use.
- Do not reuse, as device will no longer be sterile after initial use, and could result in infection.

STERILE: Sterility guaranteed unless individual wrapper is opened or damaged.

STORAGE: For maximum shelf life, storage temperature of 15-30 °C (59-86 °F) is recommended.



Not made with natural rubber latex

Symbols Used on Labeling

	Single use		Company address
	Use by date		Do not use if package is damaged
	Reference number		Method of sterilization: irradiation
	Batch number		Do not resterilize
	See instructions for use		
	Temperature Limits		



Manufacturer:

LifeScience PLUS, Inc.
P.O. Box 60783, Palo Alto, CA 94306
Toll Free: 1-877-587-5433 - U.S.A.
1-650-565-8172 - International
Fax: 1-650-336-1130
www.LifeSciencePlus.com
Email: info@lifescienceplus.com
Manufactured in the USA

LifeScience PLUS
REVOLUTIONARY WOUND CARE

Distributor:

Thomas Jane Medical
520 S. Holland St Suite 307
Wichita, KS 67209, USA
Phone: 316-251-0920
Fax: 316-251-0950
Email: purchase@sealkproducts.com
www.sealkproducts.com

Thomas Jane
MEDICAL

© 2023 LifeScience PLUS, Inc. All Rights Reserved.